

USER GUIDE

SCHAMPI CUSHION



TABLE OF CONTENTS

1	Safety instructions	4
2	Quality	5
2.1	CE marking	5
2.2	Quality work	5
3	Products covered by this user guide	5
4	Product information	5
5	Use of product	6
5.1	Receiving and unpacking	6
5.2	Applications	6
5.3	Positioning examples	6
5.4	Care and cleaning	7
5.5	Storage	7
5.6	Lifespan	7
5.7	Recycling	7
6	Other information	8
6.1	Product liability	8
6.2	Returns and complaints	8
6.3	Damage upon delivery	8
7	Markings	9
7.1	Labels	9
7.2	Symbols	9

SAFETY INSTRUCTIONS

Read the following information before using the product!

- The product should only be used in accordance with this user guide.
- The product should not be assembled, combined or repaired with any parts or accessories other than those described in this guide or in other documentation from Järven Health Care.
- Avoid using sharp objects near the product.
- In order to avoid damage, the product should not come into contact with heat sources or fire, such as burning cigarettes, heat lamps etc.
- No other plastic should come into direct contact with the cushion's surface layer, as the risk of migration is high (migration = transfer of plasticisers, colours etc. between plastics). Migration can lead to dehydration of the cushion's plasticiser coating and thereby cause cracks.
- Make sure that you use the correct product for the appropriate purpose.
- The Schampi cushion may only be used on beds fitted with a bed rail.

2. QUALITY

2.1 CE MARKING

The **Schampi Cushion** is CE marked according to Medical Device Regulation MDR (EU) 2017/745 class 1. The CE marking means that Järven Health Care has the manufacturer's responsibility to ensure that this product meets the essential requirements of relevant European legislation. This includes the provision of technical documentation and assurance which is maintained for at least ten years after the product is placed on the market.

2.2 QUALITY WORK

Our quality standards and customer focus goals are realised by responding to the needs and expectations of our customers. We believe in building long-term relationships through collaboration and trust.

Continuous improvement in our products and services is the basis of our ability to satisfy both current and future customers. Our approach includes the development of evaluation methods and standards that offer the customer a guaranteed quality in terms of comfort, function, durability and price.

Through our team of skilled and dedicated employees, we are focused on supporting education, development and innovation within the healthcare profession. Our goals are to facilitate high quality care aimed at preventing pressure ulcers, to reduce healthcare related infections and to improve fire safety within healthcare environments.

3. PRODUCTS COVERED BY THIS USER GUIDE

- SCHAMPI CUSHION (*SCHAMPIKUDDEN*)
- SCHAMPI CUSHION (*SCHAMPIKUDDEN*) small

4. PRODUCT INFORMATION

In places such as homes for the elderly, it is quite a common occurrence that a person wants to sit on the edge of the bed, but is unable to. The Schampi cushion has been designed to solve this issue. The Schampi cushion is placed behind the back of the person, once they have been helped into a sitting position. It is prevented from sliding backward by raising and securing the side rail of the bed. It provides safe and comfortable support thanks to the contoured back. The cushion is supplied with a cover made of polyurethane (PU). The cover is soft, comfortable and easy to clean.

Art no.	Size (LxWxH)	Description	Colour
13-000100-001	85x70x40 cm	Low back with armrests, PU cover	blue/beige
13-000100-002	85x58x53 cm	Narrower variant with high back, PU cover	blue/beige
13-000000-001	38x50x40 cm	Low back without armrests, PU cover	blue/beige

**The length measurement includes the armrests. The armrests are 20 cm high.*

5. USE OF PRODUCT

5.1 RECEIVING AND UNPACKING

Start by checking that the product is not damaged. In the event of transport damage, see page 7. Remove the packaging and any protective material. **Use of knives should be avoided when unpacking and handling the product due to the risk of damage.** The Schampi cushion can be used immediately after unpacking.

5.2 INTENDED USE

In places such as homes for the elderly, it is quite a common occurrence that a person wants to sit on the edge of the bed, but is unable to. The Schampi cushion has been designed to solve this issue.

The Schampi cushion is intended for use in all types of hospitals, nursing homes, assisted living apartments, special accommodation, palliative care units and units focused on rehabilitation (for example, for individuals with breathing and circulation difficulties). It can also be used in the areas of disability care, shared housing, home health care and maternity care.

5.3 POSITIONING EXAMPLES



5.4 CARE AND CLEANING

- Regularly check the cushion (and cover) for visible signs of damage or wear, such as tears, cracks, holes, stains or discolouration.
- The surface layer is cleaned with soap-based cleaning agents. Make sure to use the correct dilution ratio as recommended by the detergent instructions. Never use concentrate.
- Chlorine-containing products must **NOT** be used!
- Disinfection is done with alcohol-based cleaning agents containing surfactants.
- No other plastic should come into direct contact with the cushion's surface layer, as the risk of migration is high (migration = transfer of plasticisers, colours etc. between plastics). Migration can lead to dehydration of the cushion's plasticiser coating and thereby cause cracks.
- Fabric conditioner/softener destroys the waterproofness of the surface layer and cover and must **NOT** be used.



When using detergents or other chemicals, it is important to observe the instructions and safety recommendations for each detergent. Read the safety data sheets of the detergents used.

5.5 STORAGE

Store the Schampi cushion in the dry and at room temperature. Avoid direct sunlight.

5.6 LIFESPAN

Always follow our care instructions for the Schampi cushion to ensure the longest possible service life while maintaining function. The expected service life is 5 years.

5.7 RECYCLING

The Schampi cushion is completely energy-recoverable and can be sorted as "Combustible waste". No disassembly is necessary.

6. OTHER INFORMATION

6.1 PRODUCT LIABILITY

Product liability legislation (EU Directive 85/374/EEC, Swedish Law SFS 1992: 18) establishes the manufacturer's, importer's or seller's statutory liability for any damage caused by the product. The regulations apply to physical products in the form of movables, even if incorporated into another movable or into an immovable. If there is no product which has been the cause of some form of property or personal injury, then product liability legislation is not relevant.

6.2 RETURNS AND COMPLAINTS

Any complaints regarding the product must be notified in writing to Järven Health Care. The return of products must be done in accordance with our instructions. Returning a product before a complaint case is processed will result in the case being considered closed and customer compensation will not be paid.

6.3 DAMAGE UPON DELIVERY

Any damage of the product must be noted on the accompanying delivery document before accepting possession. Then contact Customer Service at Järven Health Care (+46 660-22 28 50, healthcare@jarven.se) and provide relevant information.

NOTE! For faster processing of your case, we ask you to photograph the transport damage as thoroughly as possible before contacting Customer Service.

7. MARKINGS

7.1 LABELS

Below is an example of how the products are labelled. The appearance of labels may vary.



7.2 SYMBOLS



Article number



Serial number



Batch number



The product is a
medical product




Consult user guide



Date of manufacture



Manufacturer

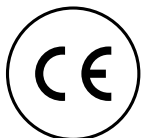

This product is CE
marked according to
Medical Device
Regulation MDR (EU)
2017/745 class 1



This product is water
repellent



Cleaning should be
undertaken with soap-
based cleaning agents




Disinfection should
be undertaken with
alcohol-based or similar
surfactant-based
detergents



No chlorine-containing
preparations may be
used!



Do not iron

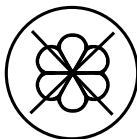

Fabric conditioner must
NOT be used

NOTES

.....

.....

.....

.....

.....

.....

.....

.....

.....

.....

NOTES

.....

.....

.....

.....

.....

.....

.....

.....

.....

.....



Schampi cushion user guide. All trademarks are the property of their respective owners.

Järven AB reserves the right to update this guide if necessary
and/or on finding any spelling or factual errors.

**Additional copies can be ordered from Järven Health Care.
Copying for internal use is allowed.**



Järven AB
Bromsvägen 3
SE-891 60 Örnsköldsvik
SWEDEN

Tel: +46 660-22 28 50
Fax: +46 660-824 10
Email: healthcare@jarven.se

www.jarven.se