

User Manual

Järven Rescue Mattress



TABLE OF CONTENTS

1. Safety information	4
2. Quality	5
2.1 CE-marking	5
2.2 Quality work	5
3. The User Manual applies to the following products	4
4. Product information	6
4.1 Järven Rescue Mattress	6
4.2 Fire protection	6
4.3 Receiving and unpacking of your new rescue mattress	6
5. Intended use	6
5.1 Evacuation with Järven Rescue Mattress	7
5.2 Care and cleaning of the rescue mattress	8
5.3 Storage	9
5.4 Lifespan	9
5.5 Recycling	9
6. Other	9
6.1 Product Liability	9
6.2 Returns and complaints	9
6.3 Transport damage	9
7. Marking	10
7.1 Labels	10
7.2 Symbols	10

SAFETY INSTRUCTIONS

Read the instructions before using our products!

- The product should only be used in accordance with the description in this manual.
- The product must not be combined, assembled or repaired with other parts, accessories or spare parts than those described in these operating instructions or other documentation from Järven Health Care.
- The product must be used so that it is not pinched or damaged.
- Avoid using sharp objects near the product.
- Järven Rescue Mattress has been fire tested. It is the customer's responsibility to make a risk assessment based on the risk of arson or other fire risks and then choose the right product. Feel free to contact us with questions.
- To avoid damage, the product must not come into contact with heat sources or fire, such as glowing/burning cigarettes, hot lamps or the like.
- Make sure you order the right product for the right purpose.
- Evacuation personnel should have adequate training.
- After evacuation the mattress must be checked carefully for any damage.
Are you unsure? Contact Järven Health Care; 0660-22 28 50, healthcare@jarven.se
- **DO NOT CARRY THE SUBJECT ON THE MATTRESS!**

2. QUALITY

2.1 CE MARKING

Järven's Rescue Mattress is CE marked according to MDR 2017/745. The marking means that Järven Health Care has the manufacturer's responsibility and assures that the product meets the essential requirements and that there is technical documentation and insurance that must be preserved for at least ten years.



2.2 QUALITY WORK

Our quality work and customer focus is realized by being sensitive to our customers' needs and expectations and in all respects work for a collaboration based on long-term perspective and trust. Continuous improvement of our products and services is the basis of our ability to satisfy customers today and in the future. Our basic quality work also includes finding evaluation methods and standards that offer the customer a guaranteed quality in terms of comfort, function, durability and price.

Järven Health Care focuses in particular on continuously training and developing the competence of healthcare staff through competent and committed employees. It is our hope that this will lead to them, through the right product choice, conducting high-quality care with a focus on preventing pressure ulcers, care-related infections and reducing fires that occur in care.

3. THE INSTRUCTIONS FOR USE APPLY

- Järven's Rescue Mattress

4. PRODUCT INFORMATION

4.1 JÄRVEN RESCUE MATTRESS

The rescue mattress can be used for a quick and safe evacuation of people who cannot get out in fire or other danger. With the rescue mattress, one person can evacuate one person in need with a few simple steps and provides extra security for those who have an establishment where an evacuation can include injured, disabled and/or those with mobility problems.

The rescue mattress is designed with equal length sides and an underside that glides just as well on wet and dry. It is X-ray transparent and equipped with Velcro straps that provide a firm fixation. The underside reflects heat. Size 190x48x8 cm

4.2 FIRE PROTECTION

Järven Rescue Mattress is tested and fulfils relevant parts of SS 876 00 39:2012. SS 876 00 01, EN 597-1.

4.3 RECEIVING AND UNPACKING YOUR NEW RESCUE MATTRESS

Start by checking that the product is not damaged. In the event of transport damage, see section 6.3. Remove the plastic packaging and any protective material. The use of a knife should be avoided when handling the packaging and the product.

5. INTENDED USE

Järven's Rescue Mattress is intended to be used when evacuating a bedridden patient in the event of a fire or other danger, when no other evacuation option is provided.


Can be used in all care environments, but is NOT recommended for patients who are acting out or patients with self-harming behaviour as the mattress has restraining straps that can be used for injury. In case of self-harming behaviour, we recommend a rescue mattress that should be placed so that it is only accessible to staff.

Järven's Rescue Mattress is intended to drag the patient on the floor, not for lifting.

5.1 EVACUATION WITH JÄRVEN RESCUE MATTRESS

EVACUATE FROM BED

- 1 Inform the patient about what is happening.
- 2 Lower the bed all the way and lock it in a position for a safer evacuation.
- 3 Make sure you have turned the mattress right side up. The mattress is marked with "THIS SIDE UP". It makes no difference how you turn the mattress in the foot/head direction.
- 4 Use the existing sheet in the bed to turn the patient in a position so that you can lay the rescue mattress under the patient.
- 5 Then pull the sheet with the patient up on the rescue mattress.
- 6 Use the tension band to attach the mattress around the patient.
- 7 Use the outer handles at the foot end to pull the mattress out of the bed and onto the floor.

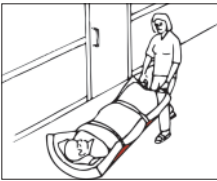

EVACUATION ON A LEVEL SURFACE

- 1 Pull the mattress with the handles at the foot end.
- 2 Tilt your body back slightly. This facilitates the work and relieves the arms.
- 3 Go at normal speed - do not run!



EVACUATION IN STAIRS

- 1 Go backwards until most of the mattress has slid down over the top step.
- 2 Turn completely so that you have your eyes in the direction of travel, without letting go of the mattress.
- 3 Hold one hand firmly on the handrail and pull with the other hand.
- 4 Follow the stairs down and let the mattress stop against your legs if necessary.
- 5 Go down with at a normal speed - do not run!



Evacuation personnel should have adequate training.

5.2 CARE AND CLEANING OF THE RESCUE MATTRESS



After evacuation, all seams must be carefully checked.

Are you unsure? Contact Järven Health Care; 0660-22 28 50, healthcare@jarven.se

Dry off the mattress with soap based cleaning products and water. Be sure to use the correct dilution according to the detergent recommendations. Never use concentrate.

Chlorine-containing preparations must not be used!
Disinfect with alcohol-based detergent with surfactants.



When using cleaning agents or other chemicals, it is important to observe the instructions and safety instructions available for each cleaning agent, see safety data sheets for the cleaning agents used.



Cleaning should be done with soap-based cleaners. Follow the cleaner's instructions for use and recommendations regarding safety and personal protection, etc. Be sure to use the correct dilution ratio. Never use concentrate.



Under no circumstances may chlorine-containing preparations be used!



Disinfect with alcohol-based or similar cleaning agents with surfactants.
NOTE! Alcohols have a dehydrating effect and should therefore be used sparingly.
These can be used; Dax Surface Disinfection Plus, Virkon, Perform and Dismozon Pur.



Avoid sharp objects near the rescue mattress.



Autoclaving is not recommended since it ages the mattress prematurely.



If odours remain after the mattress is cleaned, put a little lemon essence on a damp rag and dry off the mattress.

5.3 STORAGE

Store the rescue mattress in a dry, room temperature space. Avoid exposing the rescue mattress to direct sunlight.

5.4 LIFESPAN

Always follow our care instructions on the rescue mattress for the longest possible service life while maintaining safety and function. Estimated lifespan, if care instructions are followed, is 5 years.

5.5 RECYCLING

Järven Rescue Mattress is completely energy recyclable and must be sorted as combustible waste. No disassembly is necessary.

6. OTHER

6.1 PRODUCT LIABILITY

Product liability refers to the manufacturer's, importer's or seller's statutory liability for any damage caused by the product. The Product Liability Act (EU Directive 85/374/EEC, Swedish law SFS 1992: 18) establishes the manufacturer's, importer's or seller's statutory liability for damages for any damage caused by the product. The rules apply to physical products in the form of loose objects but also to products that are linked to other loose objects or real estate. If there is no product that has caused any form of property or personal injury, the product liability legislation is not relevant.

6.2 RETURNS AND COMPLAINTS

Complaints must be reported in writing to Järven Health Care. The product must be returned after the approved complaint has taken place in accordance with our instructions. If the product is returned before the complaint case is approved, the case is considered closed and compensation to the customer is not paid.

6.3 TRANSPORT DAMAGES


Transport damage must be noted on the shipping note before signing to be valid. You then contact customer service at Järven Health Care (0660-22 28 50, healthcare@jarven.se) and provide relevant information.

NOTE! For faster processing of your case, we ask you to photograph the transport damage as detailed as possible before you contact customer service.

7. MARKING

Below is an example of how the rescue mattress is marked. The appearance of the label may vary. The rescue mattress is marked with instructions, care instructions and the manufacturer's contact information.

7.1 LABEL

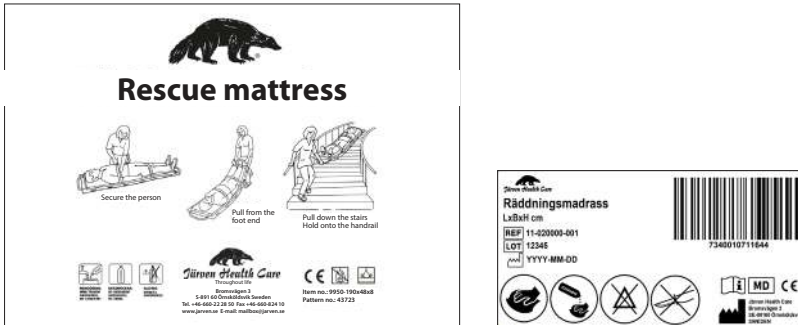

7.2 EXPLANATION OF SYMBOLS



Indicates the article number of the product



Batch number



Manufacturing date



Read the operating instructions for important warning information



This is a Medical Device



Manufacturer



Cleaning should be done with soap based preparations



Disinfection should be done with alcohol based preparations



Under no circumstances may chlorine-containing preparations be used!



Avoid sharp objects



This product is CE marked according to MDR 2017/745

7.3 ACCESSORIES

11-020100-001 Rescue Mattress sign + stand





Distributor:



Järven Health Care

Bromsvägen 3
SE-891 60 Örnsköldsvik
SWEDEN

Phone: 0660-22 28 50
Telefax: 0660-824 10
E-mail: healthcare@jarven.se

Rescue Mattress User Manual All trademarks are the property of their respective owners.

Järven AB reserves the right to update these operating instructions if necessary and / or to find any spelling or factual errors.

**Additional copies can be ordered from Järven Health Care.
Copying for internal use is permitted.**