



THE JÄRVEN EMERGENCY RESCUE SHEET

With a Järven Emergency Rescue Sheet between the mattress and bed, you can quickly evacuate the patient in the event of a fire or other emergency. By simply taking hold of the sheet, a person can evacuate the patient single-handed.

By pulling the extended handle at either end of the Järven Emergency Rescue Sheet, care personnel can evacuate the patient more ergonomically and with less physical effort during an emergency. The straps are attached by velcro and in a clear orange colour for added safety.

The Järven Emergency Rescue Sheet is compliant with Swedish standard SS 876 00 39:2012 and works on both wet and dry floors.

IMPORTANT We recommend that the Järven Emergency Rescue Sheet should not be used in combination with a mattress on top of a continental bed or a bed with a spring mattress.

Standard

The Järven Emergency Rescue Sheet meets Swedish Standard, SS 876 00 39: 2012, and this guarantees, for example, that the handles can withstand a load of 1,000 N (100 Kg), which corresponds to a patient weight of approx 210 kg. The sheet is designed to slide smoothly across both wet and dry surfaces.

X-ray

The product is X-ray transparent.

Marking

Marked with evacuation instructions in 8 languages, EAN code, care advice and contact details for the manufacturer. There is space for your own marking.

