



MELISSA PARAD

Preventive healthcare mattress with excellent pressure relief and comfort

Melissa Parad is a healthcare mattress in the Melissa series. It is designed to be part of the daily care of patients. The mattress is designed for the treatment of pressure ulcers up to and including category 3. The documented low pressure provides relief for protruding body parts such as shoulders and heels.

- CAREFULLY ASSEMBLED FOAM CORE. Melissa Parad is a 2-layer mattress with carefully selected foam qualities for comfort and good pressure relief.
- UP TO AND INCLUDING WOUND CATEGORY 3. Melissa Parad is designed to aid in the treatment of patients with pressure ulcers up to and including Wound Category 3.
- THREE WATERPROOF COVERS OF YOUR CHOICE. Melissa 12 is supplied as standard with a noiseless RX cover without handles. It can also be supplied with optional covers either the RXH cover which includes handles or the RXI cover which includes both handles and an integrated evacuation function.



MELISSA PARAD

DEVELOPED FOR HIGH COMFORT AND VERY GOOD PRESSURE RELIEF.

Waterproof cover with good properties

Melissa Parad comes with a cover from the RX series. A series of covers made of 2-stretch polyurethane fabric that is both breathable and noiseless. Depending on the choice of cover, the cover has zippers on three or four sides. All covers have a drip shield covering the zipper to facilitate hygiene.

Melissa Parad offers very good pressure relief

The foam core consists of two layers of carefully selected foams. Together with our RX cover this provides excellent pressure relief. With a documented low pressure (39 mm/Hg) for protruding body parts such as heels and shoulders, no separate heel zone is required.

Melissa Parad offers a integrated seat function

The mattress has been fatigue tested according to standard SS 876 00 11 with good results. The mattress offers a integrated seat function with a raised back position in bed.

6 cm hypersoft foam 45 kg/m3

8 cm high resilient cold foam 50 kg/m3

STANDARD FOR THE OBJECTIVE EVALUATION OF THE PRESSURE-RELIEVING PROPERTIES OF MATTRESSES.

In order to obtain an objective assessment of the pressure relieving properties of mattresses, RISE has developed a standard where the actual counterpressure of a mattress is measured by means of an indentational body. The measurement method is described in the Swedish standard SS 876 00 13.

Melissa Parad has been tested by RISE* according to method SS 876 00 13 and is pressure relieving up to wound category 3.

| Load capacity | Extra low | Low | Medium | High | |
|----------------------|-----------|-------------|-------------|-------------|--|
| Large indenter 300 N | < 30,0 | 30,0 - 34,9 | 35,0 - 39,9 | 40,0 - 44,9 | |
| Large indenter 600 N | < 45,0 | 45,0 - 52,9 | 53,0 - 60,9 | 61,0 - 68,9 | |
| Small indenter 40 N | < 45,0 | 45,0 - 52,9 | 53,0 - 60,9 | 61,0 - 68,9 | |

| Mattress | Large indenter 300 N | Large indenter 600 N | Small indenter 40 N |
|---------------|----------------------|----------------------|---------------------|
| Melissa Parad | 24 | 47 | 39 |

*RISE (Research Institutes of Sweden), a Swedish independent research institute - www.ri.se

PRODUCT SPECIFICATIONS

Materials6 cmhypersoft foam 45 kg/m³
8 cm high resilient cold foam 50kg/m³.Sizes200/205/210 x 80/85/90/105/120 x 14 cm
(For other sizes contact Järven Health Care.)Wound categoryUp to and including category 3.Mattress weightapprox. 11 kg depending on length and
width.

| | User weight | Up to and including 180 kg. | | | |
|----|-------------|---|--|--|--|
| | Fire safety | SS 87600 01, normalrisk (testmetod EN 597-1 & 2) | | | |
| .) | Recycling | Mattress with cover is fully recyclable. Sort as combustible waste. Disassembly is not required. | | | |
| | Marking | Year of manufacture, month, day, EAN, care instructions, contact information, standards met and bed location. | | | |

(F



Bromsvägen 3 | SE-891 60 Örnsköldsvik | SWEDEN Tel: +46 (0)660 - 22 28 50 | Fax: +46 (0)660 - 824 10 www.jarven.se | healthcare@jarven.se