

# OPTICELL 2 PRO<sup>V2</sup>



**TRUE ALTERNATING**  
FOR A MORE EVEN  
CYCLE TIME



**LOW PRESSURE**  
AUTOMATIC ADJUSTMENT  
FOR ALL USER WEIGHTS



**ALTERNATING OR STATIC**  
AIR PRESSURE



**MULTI-ZONE RELIEF**  
TO RELEASE CELLS  
EASILY WITHOUT TOOLS



**HYGIENE CONTROL**  
WITH THE OPTITEX  
4-WAY STRETCH COVER



**CABLE CHANNEL**  
TO PROTECT CABLE  
AND HOSE SET

*A fully automatic alternating air mattress system with  
our unique OptiTex<sup>TM</sup> hygiene cover.*

# OPTICELL 2 PRO V2

OptiCell 2 PRO V2 is a fully automatic pressure-relieving mattress with alternating pressure, used directly on the bed base. It is made up of air cells, Cell-on-Cell (N6 and N8) or a combination of single air cells on top of a base foam layer (N7 and Evac), and is used together with a supplied pump. OptiCell 2 PRO V2 is designed as an aid in the prevention and treatment of pressure ulcers up to and including category 4.

The mattress is supplied with a durable OptiTex™ cover and CPR valve for emergency situations.

The included pump, which can provide both static and alternating pressure, is easy to use and adjusts automatically to the users weight, height, position and movements while laying in bed.

MATTRESS MODEL	MATTRESS HEIGHT	MAX USER WEIGHT	MATERIAL AIR CELLS	MATTRESS TYPE	PRESSURE ULCER CATEGORY
N6	16 cm	230 kg	Nylon/TPU	Cell-on-Cell	4
N7	18 cm	230 kg	Nylon/TPU	Air/Foam	4
N8	20 cm	250 kg	Nylon/TPU	Cell-on-Cell	4
Evac	20 cm	230 kg	Nylon/TPU	Air/Foam	4

## Functions - Pump

**COMFORT FUNCTION** with six different comfort modes.

**CARE MODE** with automatic revert after 30 minutes.

**TRANSPORT MODE** the mattress can be placed in transport mode and retains maximum air pressure for at least 12 hours.

**KEY LOCK** to reduce the risk of unauthorised changes to the pump's operation.

**CPR VALVE** – In an emergency situation where CPR needs to be initiated, the CPR valve is opened to provide rapid air release.

**EXTRA LONG POWER CABEL** Comes with a 6 metre cable.

## Functions - OptiTex™ cover

**INTEGRATED CABLE HOLDER** reduces the risk of bacteria spreading and protects air hoses and electrical cables.

**FACILITATES GOOD HYGIENE**, The entire cover can be machine washed at 95°C. Cells and tubes can be easily removed from the cover and rinsed clean with warm water or disinfectant.

**ANTI-SLIP BASE WITH FOUR FASTENING STRAPS.**

**DRIP-PROOF, WATERPROOF ZIP** on four sides with welded seams.

**FOUR HANDLES** for easy transport of the mattress.




**PUMP WITH INTUITIVE DESIGN**

*The pump panel is easy to use and can be easily cleaned to reduce the risk of infection.*



**MULTI-ZONE RELIF**

*All cells can be released with ease, without tools, for total unloading.*

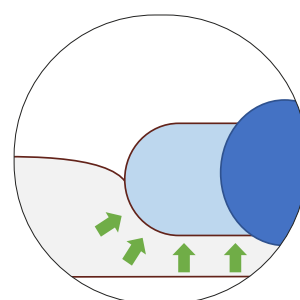

**IMMERSION & ENVELOPMENT**

*the two most important biomechanical aspects to reduce the risk of pressure ulcers*

## OPTICELL 2 PRO<sup>®</sup> EVAC with integrated evacuation function

OptiCell 2 PRO<sup>®</sup> Evac has been designed to meet the requirements of healthcare professionals to safely evacuate patients while laying on an air mattress. Especially patients who would find it difficult to escape during an emergency by themselves. An evacuation can take place safely and Single handedly thanks to the well thought out design.

**Tested and complies with SS876 00 39:2022.**

**STANDARD SIZES** 200/210/220 X 80/85/90/105/120 x 16/18/20 cm. ■ **CELLS** The mattress has removable cells made of soft nylon/TPU

**HYGIENIC COVER** OptiTex™ 4-way hygiene cover in soft, stretchy PU. ■ **PUMP** OC20, an advanced pump that adapts to user weight.

**SOUND LEVEL** 21 db. ■ **FIRE STANDARD** SS 876 00 01, EN 597-1 and 2. ■ **WEIGHT** Mattress (200 x 90 cm) 9 kg, pump 3,4 kg.



Bromsvägen 3 | SE-891 60 Örnsköldsvik | SWEDEN  
Phone: +46 (0)660 - 22 28 50 | Fax: +46 (0)660 - 824 10  
www.jarven.se | healthcare@jarven.se